

## Points of wheelchair cleaning

### 【Cleaning cushions】

- The cushions and seats are made of cloth, so remove dirt with a dry rag. If moisture is left in the cloth, it will become moldy during transportation. Take care not to use a damp rag.
- If there is any mold, use the mold removing spray and remove it completely.

cushions

belt



### 【Cleaning belts】

- Most of Velcro on the belt has mold or waste thread and they can be removed with a large-sized iron brush. Use the brush vertically and horizontally several times until the Velcro becomes clean.
- If there is any mold, use the mold removing spray and remove it completely.

金ブラシ・大



### 【Cleaning metal parts】

- Use a damp rag for coated metal parts and wipe off dirt. Take care not to use an iron brush, as it will remove coating.
- Use an iron brush for non-coated metal parts and remove stain and dirt. They will become as good as new.

### 【Cleaning main wheels】

- Use a damp rag for rim and spoke for cleaning.
- Do not use an iron brush, as it will scratch them.

Rim and spoke

金ブラシ・細

HUB



- Use a thin iron brush to remove stain and dirt on the hub of the main wheel.
- You can remove a cover, if the wheel has one, with a bit of strength and it will make it easier to clean the wheel closely.

Wheel cover



### 【Cleaning front wheels】

- Remove hairs tangled around the axle closely.
- Clean the entire wheel with a damp rag.

